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Theramon Cloud

ORTHOSMART B.V.
23 JANUARI 2019

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Theramon Cloud

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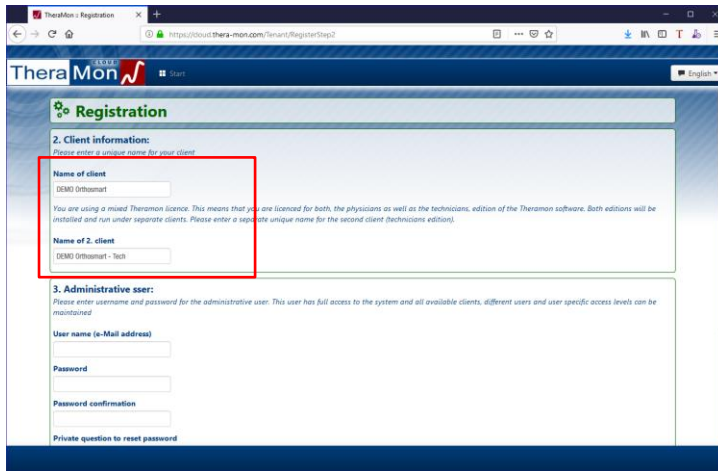
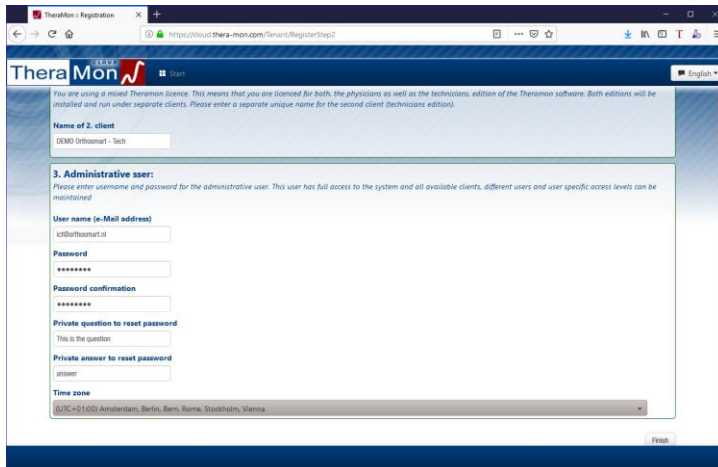
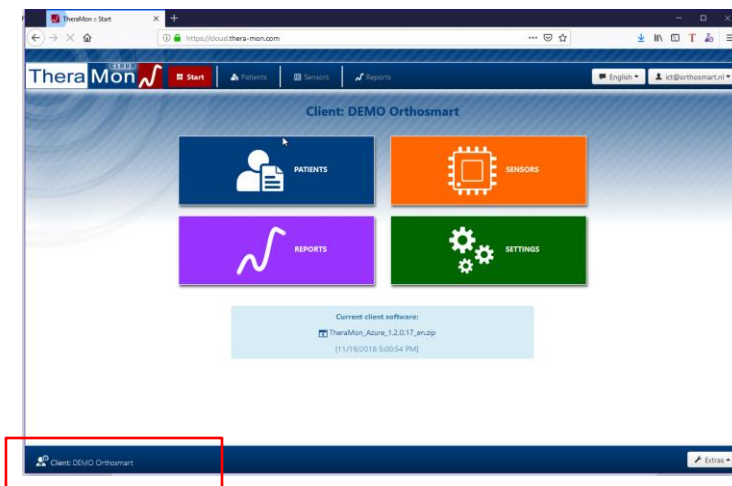
The screenshot shows the 'Registration' page for TheraMon. Under the heading '2. Client information', there are two text input fields. The first field, 'Name of client', contains the text 'DEMO Orthosmart' and is highlighted with a red rectangular box. The second field, 'Name of 2. client', contains the text 'DEMO Orthosmart - Tech'. Below these fields, there is a section for '3. Administrative user' with fields for 'User name (e-Mail address)', 'Password', 'Password confirmation', and a 'Private question to reset password'.

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The screenshot shows the 'Registration' page for TheraMon, step 3: 'Administrative user'. The 'User name (e-Mail address)' field contains 'test@orthosmart.nl'. The 'Password' and 'Password confirmation' fields both contain '*****'. The 'Private question to reset password' field contains 'antwoord'. There is also a 'Private answer to reset password' field which is empty. At the bottom, there is a 'Time zone' dropdown menu set to '(UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna' and a 'Finish' button.

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The screenshot shows the 'TheraMon Start' page. At the top, it says 'Client: DEMO Orthosmart'. Below this, there are four large buttons: 'PATIENTS' (blue), 'SENSORS' (orange), 'REPORTS' (purple), and 'SETTINGS' (green). Below the buttons, it says 'Current client software: TheraMon_Azure_1.2.0.17_an-app (11/18/2018 5:02:54 PM)'. At the bottom left, there is a 'Client: DEMO Orthosmart' label with a red box around it, and an 'Exit' button at the bottom right.

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